

Manual Supplement

Manual Title: INCU™ II Users
Part Number: FBC-0091
Print Date: October 2015
Revision/Date: 2, 1/16

Supplement Issue: **2**
Issue Date: 12/17
Page Count: 2

This supplement contains information necessary to ensure the accuracy of the above manual.

FLUKE®

Biomedical

Change #1, 311

On page 58, replace Table 15 with:

Table -15. Troubleshooting

Symptom	Resolution
The Analyzer does not show the Top Menu. (Does not turn on).	Connect the Analyzer to Power and make sure the battery is charged.
	Press and hold Analyzer power button until product resets (12 seconds).
USB/Wireless connection not working	Verify that the Analyzer is on the Main Menu
The Analyzer fails during the initial self-test.	Contact Fluke Biomedical Technical Support
The readings are inaccurate.	Make sure the probes are plugged into the correct plug.
	Make sure the probe calibration factors are correct.
The Analyzer screen does not respond.	Press and hold Analyzer power button until product resets (12 seconds).

Change #2, 536

For Russia only: на стр. 62, замените “Воздушный поток” с:

Воздушный поток

Погрешность $\pm(10 \% + 0,1)$ м/сек